

# U-MASK

## DONNING AND DOFFING



1  
Cup the mask in your hand.  
Open the nose-wire.




2  
Put the mask on your face  
starting at your chin.



3  
Secure the bottom headstrap  
around your neck below your  
ears. Then secure the top  
headstrap above your ears.



4  
Properly shape the nose-wire to  
match your face. Move the mask  
smoothly (up/down, left/right)  
until it's optimally placed on  
your face.

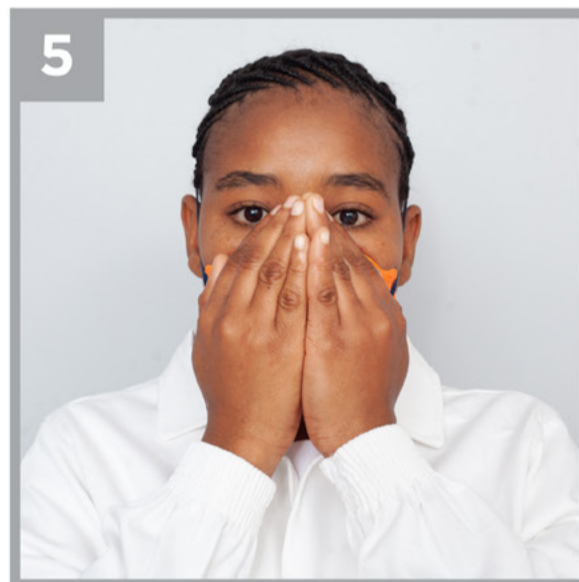

5  
Check for gaps/leaks by placing  
both hands over the mask and  
inhale sharply. If air moves around  
the edges, adjust your nose-wire  
and/or tighten your headstrap.



6  
**REMEMBER THAT  
YOUR MASK IS  
ONLY AS GOOD  
AS IT FITS!**



**Universal Safety Products (Pty) Ltd.**

U-MASK BUILDING, 288 Barolong Street, Icon  
Industrial Park, Sunderland Ridge, Centurion, Gauteng

admin@u-mask.co.za  
Tel: 010 900 0573

**www.u-mask.co.za**

